

## REUSABLE HYGIENE FACE MASKS


### UNE 0065:2020

The reusable hygiene face mask is a product for personal use which covers the mouth, the nose, and the chin, and has two elastic strip bands to place round both ears. They are made with reusable material which allows users to breath comfortably.

This mask is especially designed for adults and children 12 and over.

#### CHARACTERISTICS

- Recommended size for adults.
- Dimensions 180x95 mm.
- Composition:  
TST 50 gr/m<sup>2</sup> (100% Polypropylene).
- Elastic strips material:  
50% Polyester / 50% Latex.
- Use the elastic strips to put the reusable hygiene mask on.

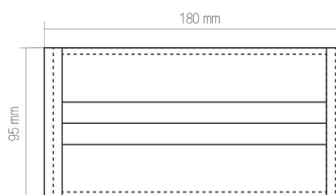

#### TESTING COMPLIANCES

UNE 0065:2020 referring to Reusable Hygiene Faces Masks for adults and children.

The product can be washed 5-7 times at 60° degrees (UNE-EN ISO 6330 6N washing cycle programme) and air-drying (UNE-EN ISO 6330:2012, A TYPE) maintaining its grades of protection.

#### WARNING

This is not a sanitary product (SP) following Directive 93/42 and regulation UE/2017/745. This is neither product for personal protection (IPE) following regulation UE/2016/425.

#### Reusable hygiene face mask acceptance criteria (following test regulation standards UNE-EN 14683):

##### TESTING

##### Bacterial filtration efficiency. (%) (Testing BFE)

Tested Bacterial efficiency (BFE) (%) (Section 5.2.2 from regulation UNE-EN 14683:2019+AC:2019)

≥ 90

##### Breathability (Differential pressure) (Pa/cm)

(Section 5.2.3 from regulation UNE-EN 14683:2019+AC:2019)

< 60

##### Acceptance criteria



## INSTRUCTIONS

We recommend putting the reusable hygiene mask on clean and dry skin (avoiding contact with the user's hair, and on shaved skin for some people) following the steps below:

1. Wash your hands with water and soap or clean them with hydroalcoholic hand-rub before using the hygiene face mask.
2. Identify the top part of the hygiene face mask. Wider elastic strip bands.
3. Place the hygiene face mask on your face, covering the top of the nose and mouth.
4. Hold the elastic strip bands and place them round your ears.
5. Pull down the bottom part of the hygiene face mask covering your chin.
6. Verify the hygiene face mask covers your chin, making sure there is no gap between your chin and the face mask.
7. Make sure the hygiene face mask is placed properly. Verify there is no gap between the hygiene face mask and your skin, making sure you can breathe properly.
8. Once you have adjusted the hygiene face mask, do not touch the hygiene face mask. If you need to touch it, you should wash your hands with water and soap or clean them with hydroalcoholic hand-rub.

## HOW TO TAKE YOUR MASK OFF

You should follow these steps to avoid any contamination risk.

1. Take off your protection gloves.
2. Wash your hands with water and soap or clean them with hydroalcoholic hand-rub.
3. Use the elastic strip bands to handle and remove the hygiene face mask. The front of the mask should not be touched.
4. Wash your hands with water and soap or clean them with hydroalcoholic hand-rub.

## WASHING AND DRYING

- We recommend giving a complete washing cycle (moisten, wash, rinse), using common detergent at a temperature of 60° degrees.
- You should avoid the contact of a dirty hygiene face mask with clean clothes. The person in charge of washing it should protect himself from dirty hygiene face masks.
- Do not use products which could damage the product material or reduce its grades of protection.
- We recommend drying the hygiene face mask completely for 2 hours after washing it. Do not use the microwave to dry your hygiene face mask.
- Avoid places which facilitate the contamination of the product when drying it and storing it.
- Check the product (with gloves or cleaned hands) after washing it. If you detect that the product is damaged (less adjustable, loss of shape, wear), you must dispose it.



## DISPOSING THE HYGIENE FACE MASK

You should dispose your hygiene face mask once you have exceeded the maximum washing cycles (5-7) of the product to reduce any risk.

## HOW TO PROCEED WHEN DISPOSING

You should dispose the hygiene face mask into a rubbish bin (no manual control lid rubbish bins if possible). We recommend using two bin bags just in case the outer one gets torn.



Scan the QR code to obtain more information about this product.

